

FlexiPump™ Irrigation Tube Set Care System

Disinfect
tube sets
every shift!



DO NOT use
bleach or alcohol
base
disinfectants on
tube sets!



Size	Quantity	Product ID
24 ounces	15 bottles per case	PSC240-1
1 gallon	4 bottles per case	PSC128-1



pure
PROCESSING

How to Disinfect Tube Sets

Disinfect every shift!

- Step 1. Attach end caps on to all 3 tube leads.
- Step 2. Pour ready-to-use Pro-Spray™ Ready-to-Use Disinfectant/Cleaner into an open, clean container.
- Step 3. Place drop down tube into container with disinfectant.
- Step 4. Set speed control on Low (1 LED light will be visible next to the speed control button).
- Step 5. Press the forward flush button to begin filling tube set with disinfectant.
- Step 6. Press **STOP** when the tube set is completely full of disinfectant. Tubing should be elevated above the FlexiPump™ to ensure tube fully.
- Step 7. Disinfectant should remain inside tubing for 5 minutes (contact time of disinfectant).
- Step 8. Rinse the disinfectant out of tube set. Fill a cleaning solution container with clean water, place the drop-down tube into rinse water.
- Step 9. Remove the end caps from luer locks.
- Step 10. Press **Forward Flush 2X for a 60 second rinse**. **CAUTION:** Flush on low speed setting.