

ProWash™

Multi-Purpose Instrument Detergent and Cart Wash

Neutral pH



Robust formula for wide variety of applications.

ProWash is a premium neutral pH multi-purpose detergent combining a powerful cleaning system with effective anti-corrosives.

Versatile:

- Ideal material compatibility to meet manufacturer's instructions. Safe for use on lensed items, aluminum, anodized aluminum, copper, stainless and carbon steel. Solvent-free cleaning agents recommended for items with gaskets, o-rings and other delicate fittings.
- Non-foaming detergent is effective for use in manual/soak applications, cart washers, ultrasonic instrument washers and high impingement automatic instrument washers.

Effective:

- Formulated for use in hard water conditions. High levels of chelating and free-rinsing agents ensure best cleaning results and prevent re-deposition of soils.

Economical:

- Concentrated and cost-efficient: Dilute at 1/2 to 1 ounce per gallon depending on water quality.



ProWash Description	Case Qty	Order#
1 Gallon	4	PW128
5 Gallon Drum	1	PW050
15 Gallon Drum	1	PW150
30 Gallon Drum	1	PW300
55 Gallon Drum	1	PW550

ProWash™

Multi-Purpose Instrument Detergent and Cart Wash

Description:

ProWash is a concentrated, neutral pH, non-foaming multi-purpose detergent suitable for use in manual and automated systems including washer/disinfectors, cart washers and ultrasonic instrument cleaning devices.

Product Specifications:

Dilution Ratios:	1 oz. per gallon of water in manual and ultrasonic systems. ½ oz. to 1 oz. per gallon of water in automated washer/disinfectors based on water quality.
pH:	8.5
Active Ingredients:	Surfactants and cleaning agents, metal surface conditioning agents, chelating, anti-redeposition and anti-corrosive agents.
Temperature of Use:	In manual applications use 90° to 120°F (32° to 49°C). In automated systems follow washer manufacturer's instructions or settings. Best results obtained at 120° to 160°F (49° to 71°C).
Solution Color:	Green.
Odor:	Fresh.
Shelf Life:	3 year shelf-life, expiration date printed on product.

Product Capabilities:

ProWash effectively cleans all types of metal and plastic instruments. In addition to conventional stainless steel items, it is compatible with aluminum trays and containers, carbon steel, copper, brass and lensed items.

Non-foaming ProWash is ideal for use in high impingement cart washers. The free-rinsing formula compensates for hard water conditions and reduces spotting

ProWash is useful for cleaning ophthalmic instruments and devices when manufacturer's instructions recommend a neutral detergent formulated without enzymes.

Accessory Products:

Other products available from Certol include:



ProEZ foam™
Ready-to-Use Foaming
Enzymatic Spray



ProEZ AW Quad™
Quadruple Enzyme
Automatic Washer
Detergent



ProLube™ Concentrate
Instrument Lubricant
Concentrate



ProLube™ RTU
Instrument Lubricant
Ready-to-Use