


FlexiPump™ Irrigation Tube Set Care System

Disinfect
tube sets
every shift!

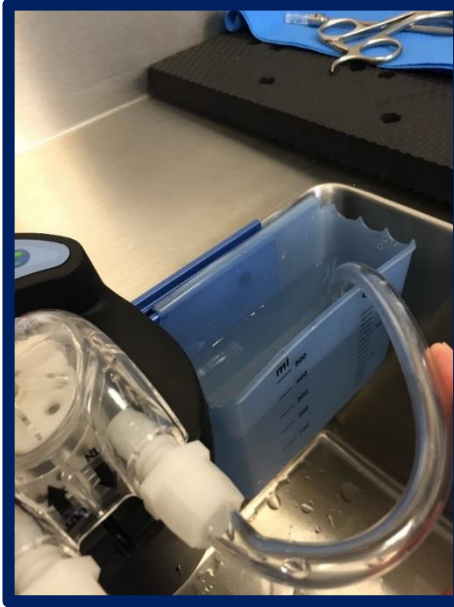

DO NOT use
bleach or alcohol
base
disinfectants on
tube sets!



Size	Quantity	Product ID
24 ounces	15 bottles per case	PSC240-1
1 gallon	4 bottles per case	PSC128-1



pure
PROCESSING

How to Disinfect Tube Sets

Disinfect after every shift!

- **Step 1.** Pour your ready to use disinfectant into an open, clean container.
- **Step 2** Place the drop down tube into the container with disinfectant, assuring its fully submerged.
- **Step 3.** Press the forward flush button to begin filling tube set with disinfectant. Run at least 100mL of disinfectant through the tube leads. Press **STOP**.
- **Step 4:** Attach end caps onto all the tube leads and set the speed control on Low. Position your pressure relief valves over a container or sink bay. Run a Forward Flush until disinfect is expelled through the pressure relief valves. Press **STOP**. Your tubes are now completely filled with disinfectant.
- **Step 5.** Disinfectant should remain inside tubing for the contact time of the disinfectant.
- **Step 6:** After the contact time has elapsed, in a empty sink basin, point the tube leads downward. Slowly take off the end caps. Some pressure will have built up inside, so use caution.
- **Step 7.** Fill a container with clean water and place the drop down tube inside. Rinse at least 500mL through the tube leads to ensure a proper rinse.