

Dear Valued Customer,

Thank you for purchasing a GelPro® Medical Mat. Whether you intend to use your mat in the operating room, at a nurses' station, or in any other room of your medical facility where there is hard flooring, we believe you will find your GelPro® Medical Mat to be a welcome addition. Our products are designed and assembled to the highest standards at our plants in Texas. For more information on your GelPro® Medical Mat, please visit www.pure-processing.com. Thank you again for your business.

CARE INSTRUCTIONS

For non-hospital environments, clean with water and a damp sponge or cloth. If you have a sticky or greasy spill you may use a mild soap (such as dish washing detergent) and water with a damp sponge or cloth. Do not use any other household cleansers such as, but not limited to, window cleaners, floor cleaners, polishes, or all purpose cleaners.

For in-hospital environments, GelPro® Medical Mats may be cleaned with a low-level, hospital grade disinfectant cleaner which kills blood-borne pathogens. We recommend using ProSpray™ Surface Disinfectant, or a cleaning agent similar in composition.

DO NOT clean with a power vacuum cleaner. The high speed rotating brush can damage the surface of the mat.

DO NOT use hot water extraction, steam cleaning, or high pressure water/air, as they could damage the mat.

DO NOT fold the mat as this can cause permanent damage.

DO NOT use the mat in wet environments or step on the mat with wet feet. The GelPro® Medical Mat is not intended for use in wet areas. The surface of the mat may be slippery when your feet are wet.

DO NOT place furniture or equipment on the GelPro® Medical Mat. Permanent indentions on mat may occur.

DO NOT allow water/liquid to become trapped under the GelPro® Medical Mat. If any moisture/liquid becomes trapped under a GelPro® Medical Mat, it may not dry and, therefore, it should be removed as soon as possible. When mopping your floor, please ensure that your floor has completely dried before replacing the mat. Prolonged exposure to moisture/liquid trapped under the GelPro® Medical Mat may cause mold/mildew and/or damage to your mat and some types of flooring surfaces.

DO NOT use acetone, window cleaners, alcohol, bleach or ammonia to clean your mat. These cleaning agents will damage your mat and void your warranty.

Customer Support

For additional questions, comments or concerns, please contact Pure Processing at 1(877) 718-6868 or e-mail sales@pure-processing.com