

Cuts-It® Gel

Instrument Pre-Soak



Micro-Scientific® Cuts-It® Gel Instrument Pre-Soak is specially formulated to safely remove blood, fat, tissue, and bodily fluids from surgical instruments and equipment. The gelling agents allow the product to adhere to instruments and soil while the blue dye makes it easy to ensure uniform coverage.

- Immediately softens and helps dissolve blood, fat, tissue and other bodily fluids
- Reduces likelihood of rust, tarnish or corrosion
- Rinses spot and residue-free

Directions for Use

Wear appropriate personal protective equipment based on safe work practices. Spray Cuts-It Gel Instrument Pre-Soak directly onto instruments immediately after the procedure or prior to the first processing step. Pay special attention to heavily soiled areas, covering instruments evenly with the blue gel. Allow instruments to soak, then rinse before processing. Instruments should be rinsed prior to entering the automatic washer.



To learn more, visit: www.pure-processing.com

For more information, call: 877.718.6868