

PureStation™ Sink Insert


CARE & CLEANING

CONSTRUCTION

The PureStation is made of out a polypropylene material, making it softer than stainless steel and lightweight. Polypropylene is safer on medical instruments than steel, and durable for long periods of use.

The PureStation™ is built to thermally insulate, maintaining enzymatic and water temperatures for longer periods during pre-cleaning.

The polypropylene is medical grade and can be wiped down with medical grade disinfectants or be put into a cart washer for easy cleaning. It is also non-porous and does not absorb liquids.

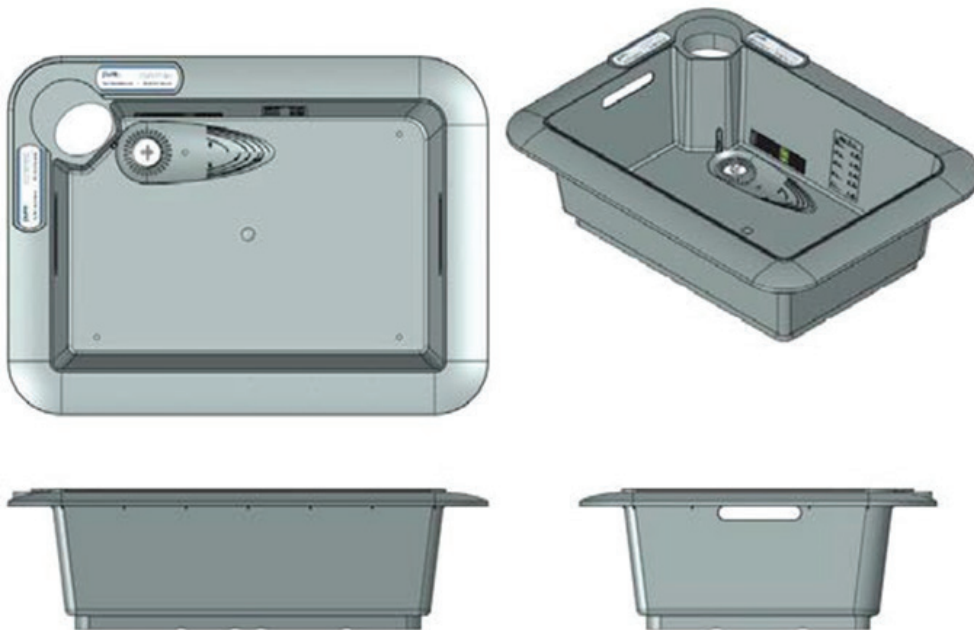

CLEANING THE PURESTATION

The PureStation has an open configuration. Simply wipe it down with the same environmental disinfectant you use on all surfaces in the department. ProSpray™ Ready to Use Disinfectant Wipes are a good disinfectant choice for cleaning the PureStation.

You may also use the PureStation in your cart washer on the low or plastic setting. The temperature of the cycle wash should not exceed 185F.