



## **PureSteel™ Mobile Soaking Station – Care and Cleaning**

### **Construction**

The PureSteel Mobile Soaking Station is a portable, stainless steel mobile cart. The PureSteel Mobile Soaking Station is intended for use in the reprocessing and disinfection of surgical equipment and devices including but not limited to leak testing, soaking, flushing, manual cleaning, and rinsing.

### **Manual Cleaning**

Spray the surface of the basin and surface lips, ensuring that coverage is consistent and surfaces remain moist. Let disinfectants rest for the full contact time per their IFU. If a volume indicator plate is in use, make sure it is also cleaned. We recommend the Spec Clean™ Ready-to-Use Disinfectant for their recommend contact times, or another intermediate disinfectant of similar composition. Surfaces can also be polished and shined with a stainless steel cleaner to prevent fingerprints and dust build-up. We recommend Weiman® Stainless Steel Cleaners for best effect.

