

PureSteel™ Task Lights

CARE & CLEANING

MANUAL CLEANING

Task lights should be disinfected routinely with intermediate level disinfectant sprays or wipes. Task lights should be turned off before being disinfected. Do not spray disinfectants directly onto bulbs.

Follow the instructions given by your disinfectant manufacturer, as contact times may vary.

It is recommended to use disinfectants low in bleach or alcohol when cleaning task lights. High concentrates of bleach or alcohol can cause fogginess or a dulling effect on magnifier lenses. A list of low bleach and low alcohol disinfectant options is provided below.



**PureSteel Halogen
Task Lights with
Magnifiers**



**PureSteel LED
Task Lights**

PRODUCT NAME	PRODUCT ID
DISINFECTANTS	
ProSpray™ 24 oz Bottle	PSC240-1
ProSpray Wipes, Soft Pack	PSW
ProSpray Wipes, Tall Cannister	PSWCT
ProSpray Wipes, Cannister	PSWC
ProSpray Gallon Bottles	PSC128-1
Opti-Cide® MAX Wipes	M60034
Opti-Cide MAX XL Wipes	M60038
Opt-Cide MAX 24 oz Spray	M60036
Opti-Cide MAX Gallon Bottle	M60035
Opt-Cide ³ 24 oz Spray	OSC12-024
Opt-Cide ³ Wipes	OCW06-100
Opt-Cide ³ 24oz Pour Bottles	OCP12-024