

Disinfecting Tube Sets

FlexiPump™ Independent Flushing System

STEP 1 Pour 500mL of your ready to use disinfectant into an open, clean container.

STEP 2 Place the drop down tube into the container with disinfectant, assuring it's fully submerged.

STEP 3 Press the forward flush button to begin filling tube set with disinfectant. Run at least 100mL of disinfectant through the tube leads. Press STOP.


STEP 4 Attach end caps onto all the tube leads and set the speed control on Low. Position your pressure relief valves over a container or sink bay. Run a Forward Flush until disinfectant is expelled through the pressure relief valves. Press STOP. Your tubes are now completely filled with disinfectant.

STEP 5 The disinfectant should remain inside tubing for the contact time of the disinfectant.

STEP 6 After the contact time has elapsed, in a empty sink basin, point the tube leads downward. Slowly take off the end caps. Some pressure will have built up inside, so use caution.

STEP 7 Run the remainder of the disinfectant through the tube sets.

STEP 8 Fill a container with 500 mL of clean water and place the drop down tube inside. Rinse at least 500mL through the tube leads to ensure a proper rinse.

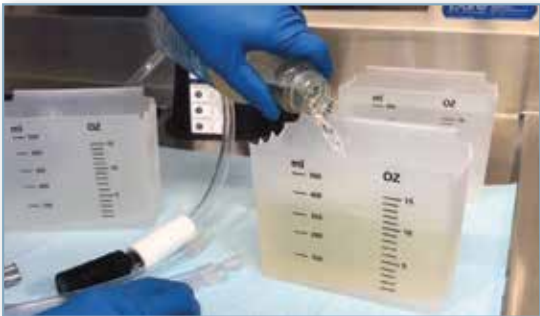

Bleach or alcohol based disinfectants are not compatible with the FlexiPump Irrigation Tube Sets.

Size	Quantity	Product ID
24 ounces	15 bottles per case	PSC240-1
1 gallon	4 bottles per case	PSC128-1

pure
PROCESSING

For more information,
call 877.718.6868 or
www.pure-processing.com