

PureSoak Timer – Installation Instructions

Items needed

- Power drill with a 5/16" drill bit appropriate for the wall
- Phillips screwdriver
- 4 ,1 ¼" Phillips screws
- 4, wall anchors appropriate for your wall



Step 1

Identify mounting holes on the back of the Timer. These will be where the screws fit. Mark the area of these (4) mounting holes on your wall.



Step 2

Using your drill bit, screw the anchors and screws into the wall where your (4) positioning marks were made. Do not screw all the way in. The Timer will sit on the screws themselves.

Step 3

Line up the timer against the (4) mounted screws. Once positioned, push the Timer down to lock it in place on the screws.