

PureSteel™ Volume Indicator Plates – Installation & Care and Cleaning

Construction


The PureSteel Volume Indicator Plates are a removable and cleanable solution for the inside of sink basins and are constructed of stainless steel.

Installation

Place the drill gauge on the edge of the sink, ensuring it is aligned with the center of the basin length. Assure that the drill gauge is on the vertical back wall of the basin (pictured below).

Use the drill punch to mark the sink for drilling the hole. Remove the hole on the back wall of the sink. Install the shoulder bracket and nut per the diagram provided with your indicator plates.

Finally, hang your volume indicator on the bolt, and test the fill of your basin for accuracy.

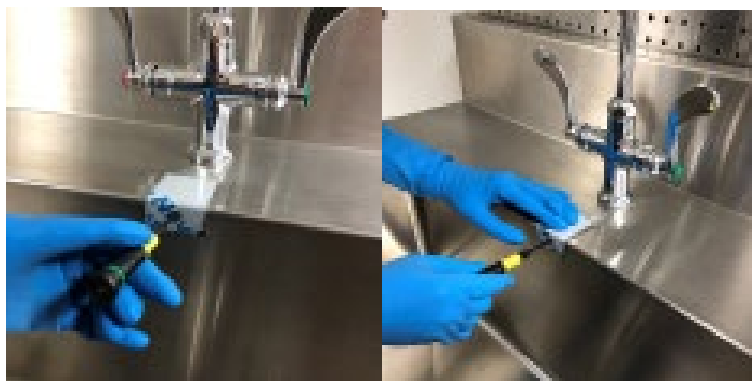

Care and Cleaning

Automated Washers

Indicator plates may be cleaned in washer/disinfectors, cart washers, or other automated washer systems on an instrument cycle, or other, similarly treated cycle.

Manual Cleaning

Spray the surface of the plate, ensuring consistent coverage of the surface. Vigorously brush the plate with a nylon toothbrush, paying special attention to the etchings. We recommend the Spec Clean™ Ready-to-Use Surface Disinfectant/Cleaner, or another intermediate disinfectant of similar composition. Indicator plates can also be polished and shined with a stainless steel cleaner to prevent fingerprints and dust build-up. We recommend Weiman® Stainless Steel Cleaners for best effect.