


XL TRANSPORT CONTAINER

XL TRANSPORT CONTAINER **INSTRUCTIONS FOR USE: English**

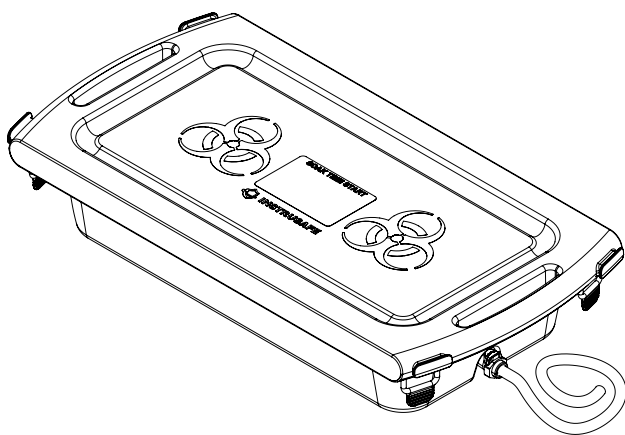

PURE
PROCESSING™

(877) 718-6868

www.pure-processing.com

Indications for Use

The container is used to collect used instruments and materials for transportation. The container can be used in a washer-disinfector and withstands repeated washing. **DO NOT** autoclave. **DO NOT** use as a sterile barrier.

Material

The container is reusable and made of high temperature plastic. It is highly durable against heat and chemicals.

Description

The container consists of a lid and base, silicone gasket and latches. The gasket and latches can be removed to help facilitate cleaning and can be easily replaced. The container base holds the instruments and materials for transportation.

Lid Removal

Release the (4) silicone latches and remove the lid.

Container Use

1. Set container on stable surface
2. Add cleaning solution per the manufacture's IFU to the container
3. Place instruments / materials in container for transportation
4. Place lid on container and engage (4) Silicone latches
5. Transport container
6. ***For IN-PIC-20-D:** To attach hose, press drain insert into coupling to open the valve
7. ***For IN-PIC-20-D:** To remove hose and close the valve, depress button and remove drain insert from coupling
8. ***For IN-PIC-20-D:** Drain should be flushed with water and air to remove any residue
9. Open lid slowly to remove instruments / materials
10. Process instruments / materials per the manufacturer's IFU

Warnings

1. **DO NOT** autoclave
2. **DO NOT** drop as this may cause damage to the instruments / materials inside or damage the container resulting in possible leakage
3. **DO NOT** shake the container with liquids inside. This can cause damage to the contents or container leakage
4. Clean and inspect prior to use. Container may be reused until mechanical failure occurs

Life Cycle: **DO NOT** use if container is cracked, broken or missing the silicone gasket or latches

5. Be cautious of pinch points while lid is being opened and closed to avoid harm to fingers and hands
6. Be cautious of container weight with liquid added
7. **DO NOT** wash container with a hard brush. This can cause damage to the container

Warnings cont.

8. Remove cover before automated washing
9. **DO NOT** stack if optional drain is installed



Manual Cleaning

1. Use a neutral (6.0 - 8.5) pH detergent
2. Remove all visible soil and contaminants using a soft brush
3. Rinse thoroughly with clean water to remove detergent


Automated Washer Cleaning

1. Container is safe for use in automated washers and can withstand up to 230°F


Gasket and latches can be replaced without the use of tools.

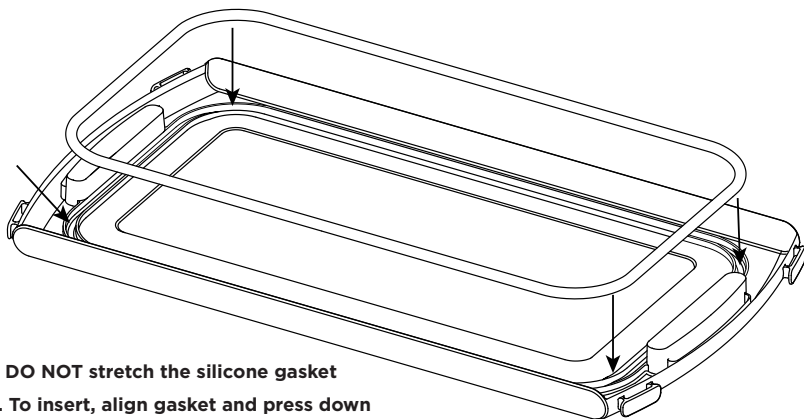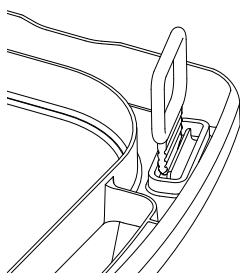Gasket Replacement**Latch Replacement**

1. Insert on top

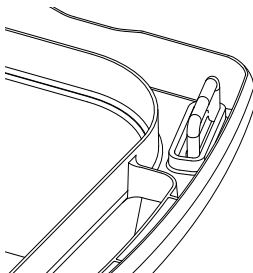

2. Position latch over groove

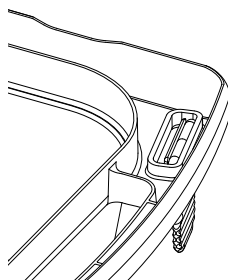

3. Pull latch down into the groove

Symbol Reference Key
Définition des symboles
Erklärung der Symbole
Clave de referencia a los símbolos
Legenda de referência dos símbolos
Legenda dei simboli
記号参照キー

	Consult instructions for use Consulter le mode d'emploi Gebrauchsanleitung konsultieren Consultar las instrucciones de uso Consultar as instruções de utilização Consultare le Istruzioni per l'uso 使用説明書を参照してください	
	Quantity Quantité Menge Cantidad	Quantidade Quantità 数量
	Catalog number Numéro de référence Katalognummer Número de referencia	Número de catálogo Numero di catalogo カタログ番号
	Lot number Numéro de lot Chargennummer Número de lote	Número de lote Numero di lotto ロット番号
	Date of manufacture Date de fabrication Herstellungsdatum Fecha de fabricación	Data de fabrico Data di produzione 製造日
	Non-sterile Non stérile Nicht steril No estéril	Não estéril Non sterile 非無菌
	Caution Attention Vorsicht Atención	Atenção Attenzione 注意

Not made with natural rubber latex.
Ne contient pas de latex naturel.
Nicht aus Naturkautschuklatex gefertigt.
No está fabricado con látex de goma natural.
Não foi fabricado com látex de borracha natural.
Non contiene lattice di gomma naturale.
天然ゴムラテックスは使用していません。



PURE
PROCESSING™

(877) 718-6868
www.pure-processing.com

