




FlexiPump™ Independent Flushing System Attaching Adapters


877.718.6868 | pure-processing.com



FlexiPump Unit

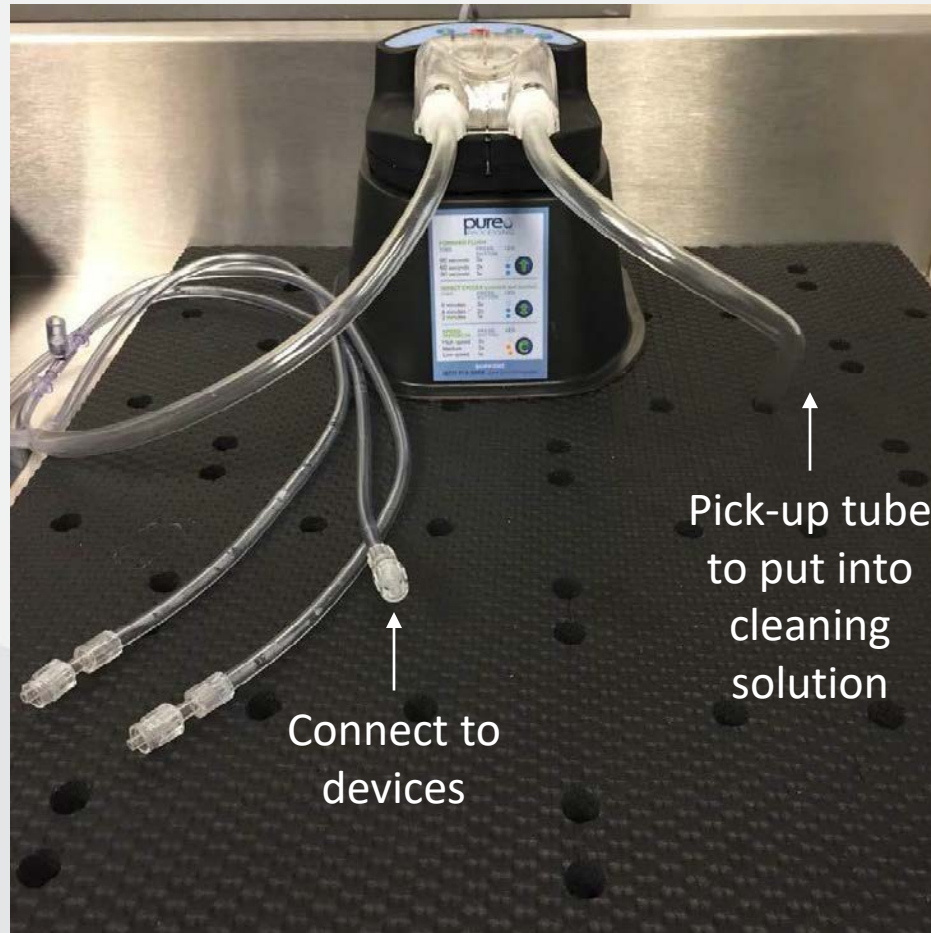


Luer locks

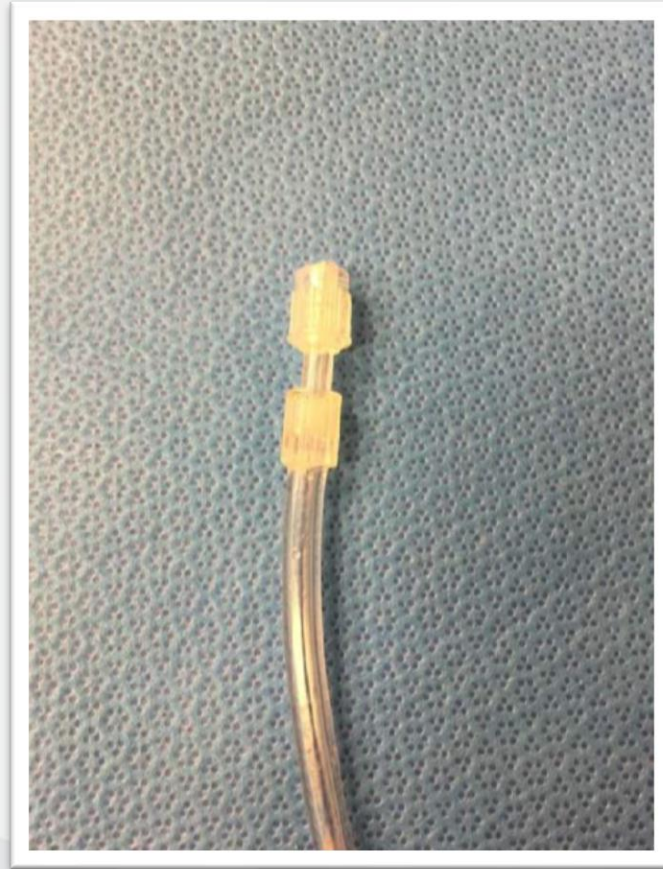

Connecting luer with devices

Easy connection with luer and slip tip

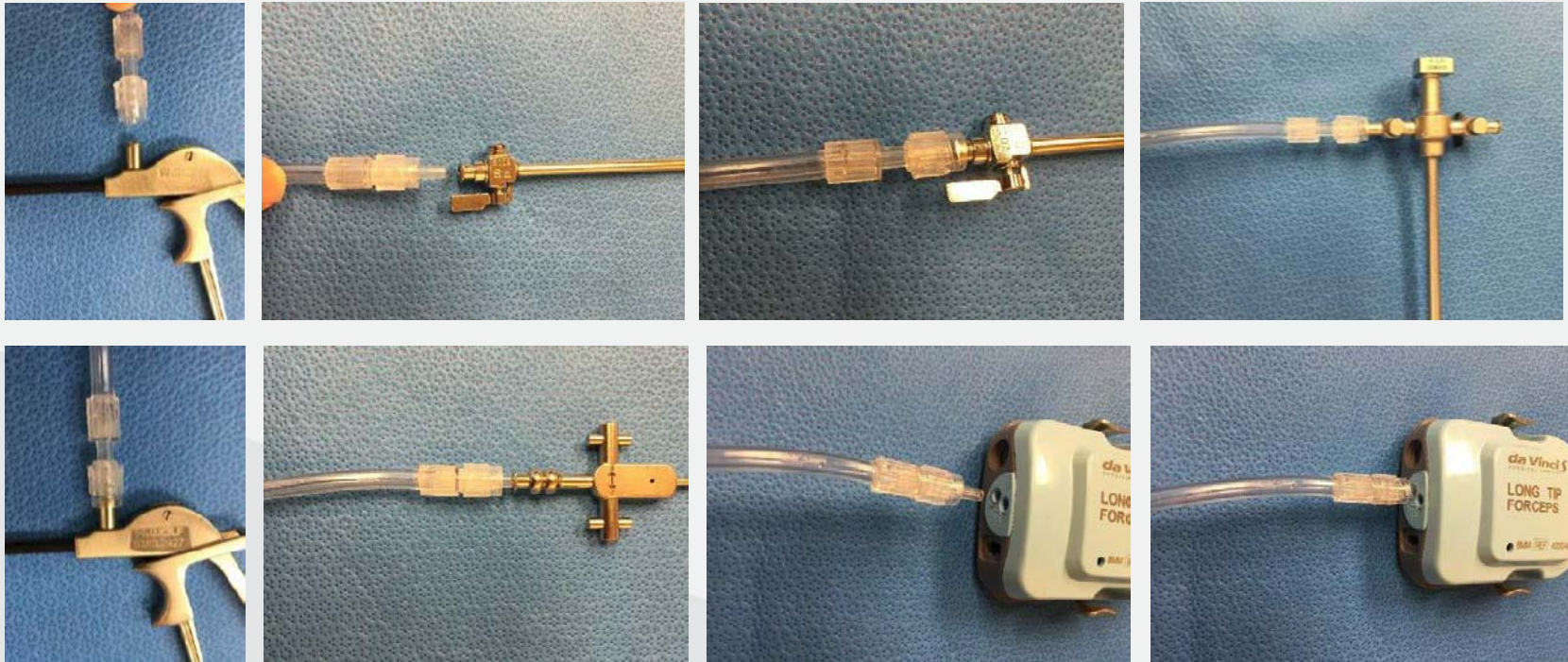

Adapters

Additional adapters available: act as the seal or bridge from the device to tube set on the pump.



**XL Slip Tip
Adapter**
Product
ID:
XLSLIP-30



**Female to
Female luer
lock**
Product ID:
FFLL-30




**Rubber Cup
Adapter**
Product ID:
RCA-01




**End cap to
seal tube.**
Product
ID: PSFP-
EC-10

Adapters

All adapters below act as the seal or bridge from the device to tube set on the pump.

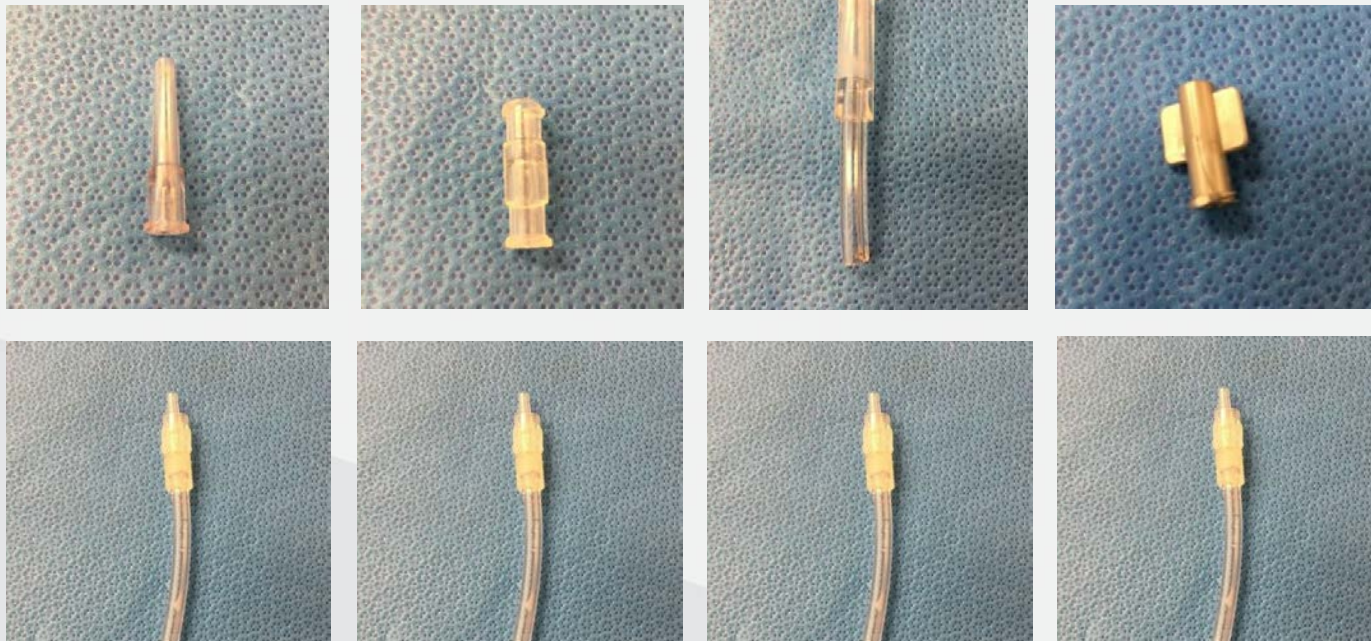


Adapters

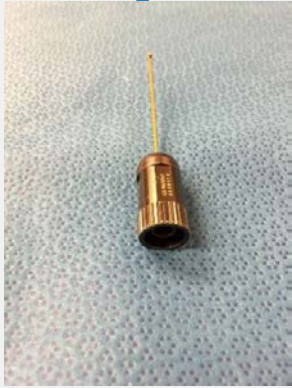

Flushing robotic ports

- Flush both ports at once; use XL Slip Tip

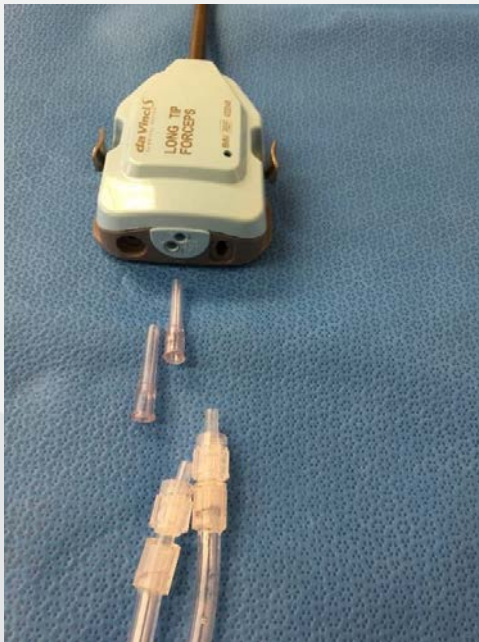

Slip tip into port



Screw in the luer,
then flush

Rubber cup adapter



Slide cup adapter
over devices end



Slide slip tip into
tube end-FLUSH

XL Slip Tip




Put XL slip tip into opening on device



Slid, twist to engage lack and get seal from tube set on pump

Female to Female Luer Lock

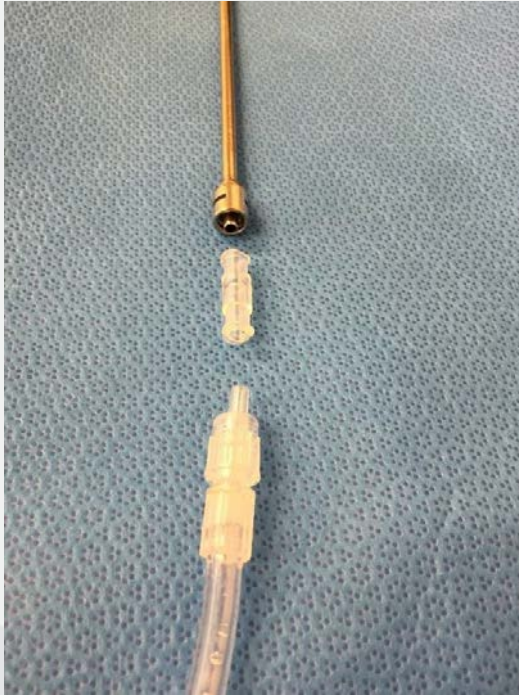

Screw the female end
into device



Screw male end of tube
set into female

Pure Processing

877.718.6868

www.pure-processing.com

sales@pure-processing.com

